Duschschemel RFM

DE

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Shower stool RFM

EN

Carefully read the complete instruction manual before installing the shower stool RFM or using it for the first time. Keep the user manual safely so that it is available for later use.

Intended purpose

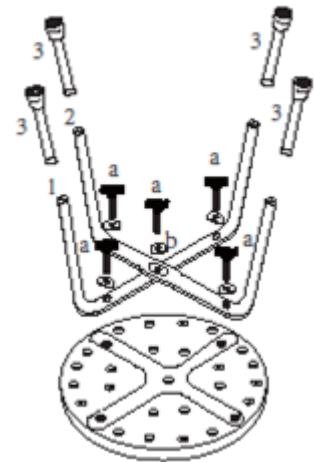
To assist in showers for those who are restricted in mobility and can no longer stand. The textured plastic of the seat surface ensures a safe hold during use. The large rubber feet prevent slipping and guarantee a stable stand. With the height-adjustable legs, the shower stool is suitable for use by people with different body lengths.

Indication

The RFM shower stool can be used for loss of lower limbs, in case of impaired vision and equilibrium, mild restriction of mobility of the legs and knee joints and loss of strength, which makes it impossible to stand for too long.

Assembly instructions - possible without tools

1. Remove all items from the box, check for completeness and damage and keep in hand.
2. Place the seating surface upside down on the table or the floor.
3. Insert the two frame parts over each other in the recesses present below the seating surface. Make sure that the frame part lies at the bottom with the recess.
4. Screw the frame parts with the 5 wing screws on the seat and place each a spacer ring below the screw.
5. Screw the legs in succession in the frame openings, making sure that these firmly click into the frame openings in the same level with the help of the push buttons.

Safety instructions

1. Ensure that the shower stool RFM stands with all 4 feet on the bottom of the shower tray before using this.
2. Make sure that all 4 legs are set at the same height and are firmly fitted in the frame parts!
3. Before each use, check the strength of the seating surface and the state of the legs!
4. If you notice any damage to the material or frame, please contact your dealer!
5. Do not use a defective shower stool RFM!
6. Do not replace or change any parts!
7. The maximum load of the shower stool RFM is 120 kg!

Cleansing

You should clean and disinfect the shower stool RFM, which is intended for reuse, at regular intervals. This is compulsory if the chair is used and re-used by another patient! The stool should be rinsed and dried after each use.

To clean, use a commercially available mild soap without abrasive additives and a dry soft cloth. You can carry out the disinfection with conventional solutions in accordance with the manufacturer's instructions. Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners! Maintain a good ventilation of the bath!

Maintenance

The shower stool RFM is maintenance-free. The frame is made of aluminum and not prone to corrosion. The seating surface is made of robust polyethylene. If you notice changes in the material or on the frame or see any signs of wear, you should contact your dealer.

If needed, use only original spare parts.

Do not use a defective shower stool RFM!

Storage

Protect the shower stool RFM against permanent moisture and place it at a safe place to protect the frame. When the shower stool is not used, the storage in the outer carton is recommended.

Contraindication

Use of the shower stool RFM is unsuitable for perceptual impairments, severe vision and equilibrium disorders - there is a risk of falling! In case of joint damage of the arms or the amputation of the upper limbs, the use is not encouraged.

Delivery contents

Shower stool
Instructions for use
Outer carton

List of parts

No.	Description	Specification	Quantity
1	Lower frame	Ø25*1.3*890	1
2	Upper frame	Ø25*1.3*890	1
3	Leg extension	Ø28*1.2*245	4
a	ScrewØ	M6 * 37	5
b	Spacer ring	Ø18 * 1.2	5

Technical specification / dimensions and weight

Maximum load: 120 kg
Diameter of seat: 32 cm
Height (adjustable): 35 - 53 cm
Material: aluminum / high density polyethylene

Guarantee

The warranty period is 24 months and shall apply to any product defects that can be attributed to manufacturing or material defect.

Daimlerstrasse 12 a
25337 Elmshorn
Tel.: +49 4121 4733 12
Fax: +49 4121 4733 23
www.rehaforum.com
info@rehaforum.com

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